

FHIR® STU 3 Conformance Statement **



****Applicable to MIM Harmony™ 1.2.x**

MIM Software Inc.
25800 Science Park Drive – Suite 180
Cleveland, Ohio 44122

216-455-0600
866-421-2536 (U.S. toll free)
www.mimsoftware.com

TD-924 (Revision Date:20 July 2021)

FHIR is the registered trademark of the Health Level 7 International for its standards publications relating to digital communications of medical information.

1 Revision History

Issue Date:	Prepared by:	Description of Change:
10/20/2020	M. Lukas	Initial Release
2/2/2021	M. Lukas	Updated Section 5 “Capability Statement Example” to match MIM Harmony 1.1.0’s version of the statement. This update to the statement contains no changes to supported resources and their interactions; it only contains minor changes to reference links for supported resources and a minor update to the 3.0.2 FHIR version from 3.0.1.
7/20/2021	M. Lukas	Updated Section 5 “Capability Statement Example” to match MIM Harmony 1.2.0’s version of the statement. This update to the statement contains no changes to supported resources and their interactions; it only contains an update to the server’s version and timestamp information, and example domain.

2 Copyright Statement

MIM FHIR Conformance Statement
 © 2003 - 2021 MIM Software Inc., All Rights Reserved.

3 *Disclaimer*

MIM Software Inc. has taken care to ensure the accuracy of this document. However, MIM Software assumes no liability for errors or omissions and reserves the right to make changes without further notice to any products herein to improve reliability, function, or design. MIM Software provides this guide without warranty of any kind, implied or expressed, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose. MIM Software may make improvements or changes in the product(s) and/or program(s) described in this guide at any time and reserves the right to revise this guide without obligation on the part of MIM Software to provide notification of such revision or change.

Additionally, while developing the MIM Harmony platform, care has been taken to ensure conformance to a subset of the interactions and resources described by the RESTful API exchange framework of version 3.0.2 (STU 3) of the Fast Healthcare Interoperability Resources (FHIR) specification published by Health Level 7 International (HL7). However, the user must perform acceptance testing to verify that the MIM Harmony platform meets the requirements for their given configuration. The acceptance testing must include all representative resources intended for transfer, all supported types of transfers desired for a type of resource, and clinical evaluation of each representative resource on the receiving end after each desired type of transfer.

4 Conformance Statement Overview

The supported subset of FHIR interactions and resources is described in the Capability Statement FHIR resource published by each MIM Harmony platform installation at its Capability Statement HTTP URL path, which is `/fhir/metadata`. The Capability Statement can be produced in either JSON or XML format by appending `?_format=json` or `?_format=xml` to the path, respectively. For example, for a server running with the IP address `192.168.1.1`, that server's Capability Statement can be accessed in JSON form at `https://192.168.1.1/fhir/metadata?_format=json`.

The Capability Statement also includes information specific to the server from which it is published, including the MIM Harmony platform version, build number, and FHIR base URL.

An example Capability Statement follows on the next page.

5 Capability Statement Example

```
{
  "resourceType": "CapabilityStatement",
  "status": "active",
  "date": "2021-07-15T15:09:01-04:00",
  "publisher": "MIM Software Inc.",
  "kind": "instance",
  "software": {
    "name": "MIM Web Platform FHIR Server",
    "version": "1.2.0-L715-05"
  },
  "implementation": {
    "description": "HAPI FHIR",
    "url": "https://ExampleDomain/fhir"
  },
  "fhirVersion": "3.0.2",
  "acceptUnknown": "extensions",
  "format": [ "application/fhir+xml", "application/fhir+json" ],
  "rest": [ {
    "mode": "server",
    "security": {
      "service": [ {
        "coding": [ {
          "system": "http://hl7.org/fhir/restful-security-service",
          "code": "Basic",
          "display": "Basic"
        } ]
      } ]
    } ]
  },
  "resource": [ {
    "type": "ActivityDefinition",
    "profile": {
      "reference": "http://hl7.org/fhir/Profile/ActivityDefinition"
    },
    "interaction": [ {
      "code": "read"
    }, {
      "code": "update"
    }, {
      "code": "delete"
    }, {
      "code": "create"
    }, {
      "code": "search-type"
    } ],
    "conditionalCreate": true,
    "searchParam": [ {
      "name": "_id",
      "type": "token",
      "documentation": "The ID of the resource"
    }, {
      "name": "_lastUpdated",
      "type": "date"
    } ]
  } ]
}
```

```

    } ]
  }, {
    "type": "Appointment",
    "profile": {
      "reference": "http://hl7.org/fhir/Profile/Appointment"
    },
    "interaction": [ {
      "code": "read"
    }, {
      "code": "update"
    }, {
      "code": "delete"
    }, {
      "code": "create"
    }, {
      "code": "search-type"
    } ],
    "conditionalCreate": true,
    "searchParam": [ {
      "name": "_id",
      "type": "token",
      "documentation": "The ID of the resource"
    }, {
      "name": "_lastUpdated",
      "type": "date"
    } ]
  }, {
    "type": "AuditEvent",
    "profile": {
      "reference": "http://hl7.org/fhir/Profile/AuditEvent"
    },
    "interaction": [ {
      "code": "read"
    }, {
      "code": "search-type"
    } ],
    "searchParam": [ {
      "name": "_id",
      "type": "token",
      "documentation": "The ID of the resource"
    }, {
      "name": "action",
      "type": "token",
      "documentation": "Type of action performed during the event"
    }, {
      "name": "address",
      "type": "string",
      "documentation": "Identifier for the network access point of the user
device"
    }, {
      "name": "agent",
      "type": "reference",
      "documentation": "Direct reference to resource"
    }, {
      "name": "date",

```

```

    "type": "date",
    "documentation": "Time when the event occurred on source"
  }, {
    "name": "entity",
    "type": "reference",
    "documentation": "Specific instance of resource"
  }, {
    "name": "entity-id",
    "type": "token",
    "documentation": "Specific instance of object"
  }, {
    "name": "outcome",
    "type": "token",
    "documentation": "Whether the event succeeded or failed"
  }, {
    "name": "subtype",
    "type": "token",
    "documentation": "More specific type/id for the event"
  }, {
    "name": "type",
    "type": "token",
    "documentation": "Type/identifier of event"
  } ]
}, {
  "type": "CareTeam",
  "profile": {
    "reference": "http://hl7.org/fhir/Profile/CareTeam"
  },
  "interaction": [ {
    "code": "read"
  }, {
    "code": "update"
  }, {
    "code": "delete"
  }, {
    "code": "create"
  }, {
    "code": "search-type"
  } ],
  "conditionalCreate": true,
  "searchParam": [ {
    "name": "_id",
    "type": "token",
    "documentation": "The ID of the resource"
  }, {
    "name": "_lastUpdated",
    "type": "date"
  } ]
}, {
  "type": "DocumentReference",
  "profile": {
    "reference": "http://hl7.org/fhir/Profile/DocumentReference"
  },
  "interaction": [ {
    "code": "read"
  } ]
} ]

```

```

    }, {
      "code": "update"
    }, {
      "code": "delete"
    }, {
      "code": "create"
    }, {
      "code": "search-type"
    } ],
    "conditionalCreate": true,
    "searchParam": [ {
      "name": "_id",
      "type": "token",
      "documentation": "The ID of the resource"
    }, {
      "name": "_lastUpdated",
      "type": "date"
    } ]
  }, {
    "type": "Endpoint",
    "profile": {
      "reference": "http://hl7.org/fhir/Profile/Endpoint"
    },
    "interaction": [ {
      "code": "read"
    }, {
      "code": "update"
    }, {
      "code": "delete"
    }, {
      "code": "create"
    }, {
      "code": "search-type"
    } ],
    "conditionalCreate": true,
    "searchParam": [ {
      "name": "_id",
      "type": "token",
      "documentation": "The ID of the resource"
    }, {
      "name": "_lastUpdated",
      "type": "date"
    } ]
  }, {
    "type": "Group",
    "profile": {
      "reference": "http://hl7.org/fhir/Profile/Group"
    },
    "interaction": [ {
      "code": "read"
    }, {
      "code": "update"
    }, {
      "code": "delete"
    }, {

```



```

    "code": "create"
  }, {
    "code": "search-type"
  } ],
  "conditionalCreate": true,
  "searchParam": [ {
    "name": "_id",
    "type": "token",
    "documentation": "The ID of the resource"
  }, {
    "name": "_lastUpdated",
    "type": "date"
  } ]
}, {
  "type": "ImagingStudy",
  "profile": {
    "reference": "http://hl7.org/fhir/Profile/ImagingStudy"
  },
  "interaction": [ {
    "code": "read"
  }, {
    "code": "update"
  }, {
    "code": "delete"
  }, {
    "code": "create"
  }, {
    "code": "search-type"
  } ],
  "conditionalCreate": true,
  "searchParam": [ {
    "name": "_id",
    "type": "token",
    "documentation": "The ID of the resource"
  }, {
    "name": "_lastUpdated",
    "type": "date"
  } ]
}, {
  "type": "Organization",
  "profile": {
    "reference": "http://hl7.org/fhir/Profile/Organization"
  },
  "interaction": [ {
    "code": "read"
  }, {
    "code": "update"
  }, {
    "code": "delete"
  }, {
    "code": "create"
  }, {
    "code": "search-type"
  } ],
  "conditionalCreate": true,

```

```

    "searchParam": [ {
      "name": "_id",
      "type": "token",
      "documentation": "The ID of the resource"
    }, {
      "name": "_lastUpdated",
      "type": "date"
    } ]
  }, {
    "type": "Patient",
    "profile": {
      "reference": "http://hl7.org/fhir/Profile/Patient"
    },
    "interaction": [ {
      "code": "read"
    }, {
      "code": "update"
    }, {
      "code": "delete"
    }, {
      "code": "create"
    }, {
      "code": "search-type"
    } ],
    "conditionalCreate": true,
    "searchParam": [ {
      "name": "_id",
      "type": "token",
      "documentation": "The ID of the resource"
    }, {
      "name": "_lastUpdated",
      "type": "date"
    } ]
  }, {
    "type": "Practitioner",
    "profile": {
      "reference": "http://hl7.org/fhir/Profile/Practitioner"
    },
    "interaction": [ {
      "code": "read"
    }, {
      "code": "create"
    }, {
      "code": "search-type"
    } ],
    "conditionalCreate": true,
    "searchParam": [ {
      "name": "_id",
      "type": "token",
      "documentation": "The ID of the resource"
    }, {
      "name": "active",
      "type": "token",
      "documentation": "Whether the practitioner record is active"
    }, {

```

```

    "name": "email",
    "type": "token",
    "documentation": "A value in an email contact"
  }, {
    "name": "identifier",
    "type": "token",
    "documentation": "A practitioner's Identifier"
  }, {
    "name": "name",
    "type": "string",
    "documentation": "A server defined search that may match any of the
string fields in the HumanName, including family, give, prefix, suffix,
suffix, and/or text"
  }, {
    "name": "role",
    "type": "token"
  }, {
    "name": "site",
    "type": "reference"
  } ]
}, {
  "type": "Questionnaire",
  "profile": {
    "reference": "http://hl7.org/fhir/Profile/Questionnaire"
  },
  "interaction": [ {
    "code": "read"
  }, {
    "code": "vread"
  }, {
    "code": "update"
  }, {
    "code": "delete"
  }, {
    "code": "history-instance"
  }, {
    "code": "create"
  }, {
    "code": "search-type"
  } ],
  "conditionalCreate": true,
  "searchParam": [ {
    "name": "_id",
    "type": "token",
    "documentation": "The ID of the resource"
  }, {
    "name": "_lastUpdated",
    "type": "date"
  } ]
}, {
  "type": "QuestionnaireResponse",
  "profile": {
    "reference": "http://hl7.org/fhir/Profile/QuestionnaireResponse"
  },
  "interaction": [ {

```

```

        "code": "read"
    }, {
        "code": "update"
    }, {
        "code": "delete"
    }, {
        "code": "create"
    }, {
        "code": "search-type"
    } ],
    "conditionalCreate": true,
    "searchParam": [ {
        "name": "_id",
        "type": "token",
        "documentation": "The ID of the resource"
    }, {
        "name": "_lastUpdated",
        "type": "date"
    } ]
}, {
    "type": "Task",
    "profile": {
        "reference": "http://hl7.org/fhir/Profile/Task"
    },
    "interaction": [ {
        "code": "read"
    }, {
        "code": "update"
    }, {
        "code": "delete"
    }, {
        "code": "create"
    }, {
        "code": "search-type"
    } ],
    "conditionalCreate": true,
    "searchParam": [ {
        "name": "_id",
        "type": "token",
        "documentation": "The ID of the resource"
    }, {
        "name": "_lastUpdated",
        "type": "date"
    } ]
} ]
} ]
}

```